


# HAIRCARE RESTRUCTURING HAIR BOOSTER

by REVITACARE

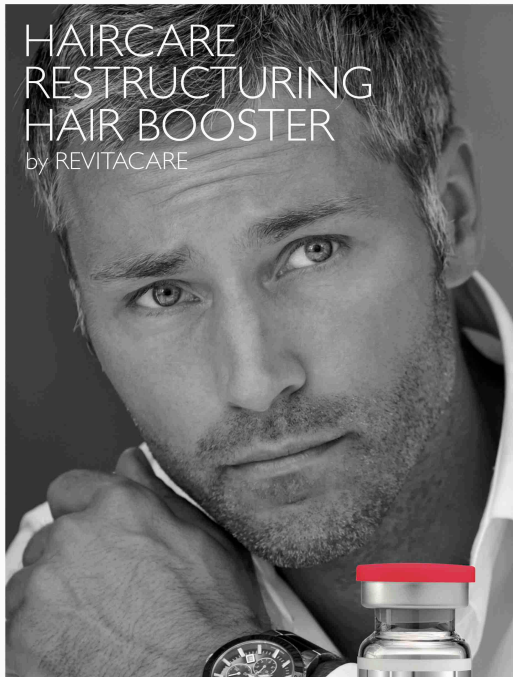


LABORATOIRE  
**REVITACARE**  
FRANCE

[www.revitacare.net](http://www.revitacare.net)



MADE IN FRANCE

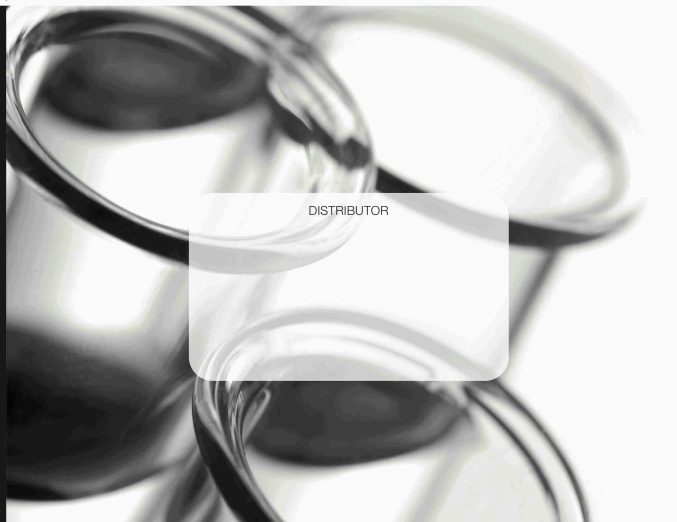

DISTRIBUTOR

REVITACARE SARL is a french laboratory, founded in 2003, specialized in research, design, manufacture of medical devices based on hyaluronic acid, dedicated to aesthetic medicine and cosmetology to prevent, correct the skin aging effects, and treat hair problems.

HAIRCARE complies with the essential requirements of the Council Directive 93/42/EEC concerning medical devices and its amendments.

QUALITY  
SECURITY  
EFFICIENCY


- REVITACARE SARL: EN ISO 13485 certified.
- Class III Medical Devices: CE0499
- HA Quality: biotechnological origin.
- Microbiological safety.
- Biocompatible in compliance with ISO 10993 international standards: Non cytotoxic - Non irritant - Non sensitizing - Non mutagenic.
- Proved efficiency by clinical data.
- Manufacturing process according to Good Manufacturing Practices (GMP).

LABORATOIRE  
**REVITACARE**  
FRANCE

Heligarc - 6 Rue Saint Simon - Parc d'Activités du Vert Galant - 95310 Saint Quentin (Aumône) - FRANCE  
Tél: +33 (0) 1 30 37 31 45 - Fax: +33 (0) 1 30 37 88 93 - Email: [contact@revitacare.fr](mailto:contact@revitacare.fr)  
[www.revitacare.net](http://www.revitacare.net)

# HAIRCARE RESTRUCTURING HAIR BOOSTER

by REVITACARE

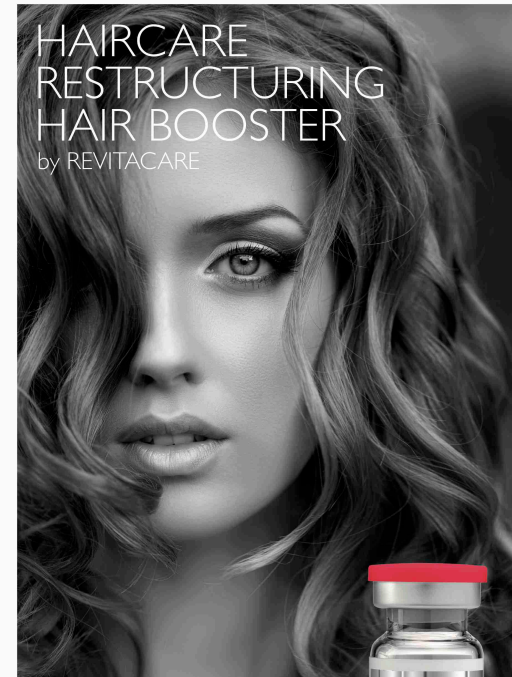

LABORATOIRE  
**REVITACARE**  
FRANCE

[www.revitacare.net](http://www.revitacare.net)



MADE IN FRANCE

REF: JARPOUVES-14-12

HAIRCARE decreases hair loss of whatever its origin\*.



CASE 1 : female patient  
After 3 weeks of treatment hair growth is visible.



CASE 2 : male patient  
After 2 months of treatment hair loss is slowed down and hair regrowth stimulated.



CASE 3 : male patient  
After 2 months hair regrowth is accelerated and quality of hair improved.

\* Visible results depend on life cycle of hair.

HAIRCARE contributes to the effectiveness of hair implantation by preparing the scalp before surgery, and anchoring the new hair into the scalp after surgery.




CASE 4 : male patient

Protocol: 3 HAIRCARE sessions one month before hair implantation, then 15 days after surgery, 1 HAIRCARE session per week during 6 weeks.

## INDICATIONS

Treatment and attenuation of the symptoms of various scalp and hair problems for both men and women:

- hair loss of whatever the origin,
- dehydration of the scalp and presence of dandruff,
- hair breakage.

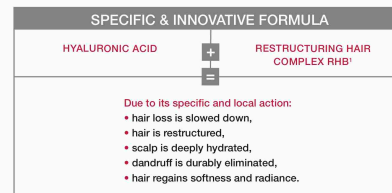

1: Buffered protective medium: Zinc Gluconate, Arginine, Cysteine, Glutamine, Glycine, Ornithine, Biotin, Calcium Pantothenate, Cyanocobalamin, Folic Acid, Nicotinamide, Pyridoxine.

## HAIRCARE ACTS ON 3 TARGETS

- **HAIR FOLLICLE**
  - Hair follicle is restructured.
  - It becomes more resistant and its anchoring is reinforced into the scalp.

⊖ Hair loss is slowed down.
- **SCALP**
  - Scalp is deeply hydrated,
  - Antioxidant protection of the dermis and purification of the scalp.

⊖ Dandruff is durably eliminated.
- **HAIR TEXTURE**
  - Hair is restructured.
  - Hair is protected against external aggression due to its antioxidant action.

⊖ Hair regains strength and radiance.

## EFFICIENCY CLINICALLY DEMONSTRATED

Multicentric clinical register on 122 patients

PRACTITIONERS SATISFACTION		PATIENTS SATISFACTION
		satisfied or highly satisfied by HAIRCARE results*
96%	regarding hair loss decrease	98%
81%	regarding scalp rehydration	77%
81%	regarding scalp cleansing	81%
99%	regarding hair texture	99%

\*Satisfaction upon protocol completion (6 sessions at intervals of 6 to 10 days)

## PROTOCOL

- Injection into the superficial dermis of the scalp, near the hair roots.
- **Needle:** 30G needle is recommended.
- **Protocol:** 6 sessions at interval of 6 to 10 days are recommended. Further to this protocol, practitioner decides on the frequency of the background therapy, according to severity of the case.

## CHARACTERISTICS

- Box: 10 vials x 5ml
- Sterilization: aseptic filtration
- Pyrogen free
- Ready to use
- Conservation condition: 2°C to 30°C

HAIRCARE is a medical device class II CE 0499, indicated in the treatment and attenuation of the symptoms of various scalp and hair problems: hair loss of whatever origin, dehydration of the scalp, presence of dandruff and hair breakage, manufactured by REVITACARE SARL. For medical use. Read carefully HAIRCARE leaflet before using the product.  
HAIRCARE must be injected in an aseptic manner, by duly practitioner, with sterile and single use equipment, which is not supplied by REVITACARE SARL. Before use HAIRCARE, check the expiry date on the labels of the vials, the integrity of the vial and the appearance of the solution. Store between 2°C and 30°C, away from the light in its original secondary packaging. Do not use if package is damaged. Once the vial of HAIRCARE has been opened use it immediately and discard any remaining product after use, complying with the procedures in force in the establishment. Single use solution: do not re-use and do not re-sterilize. The practitioner must inform the patient that there are potential adverse effects linked to the injection of our device such as pain, tingling, redness, mild local inflammation, oedema and small haematomas.

